
PROTEOR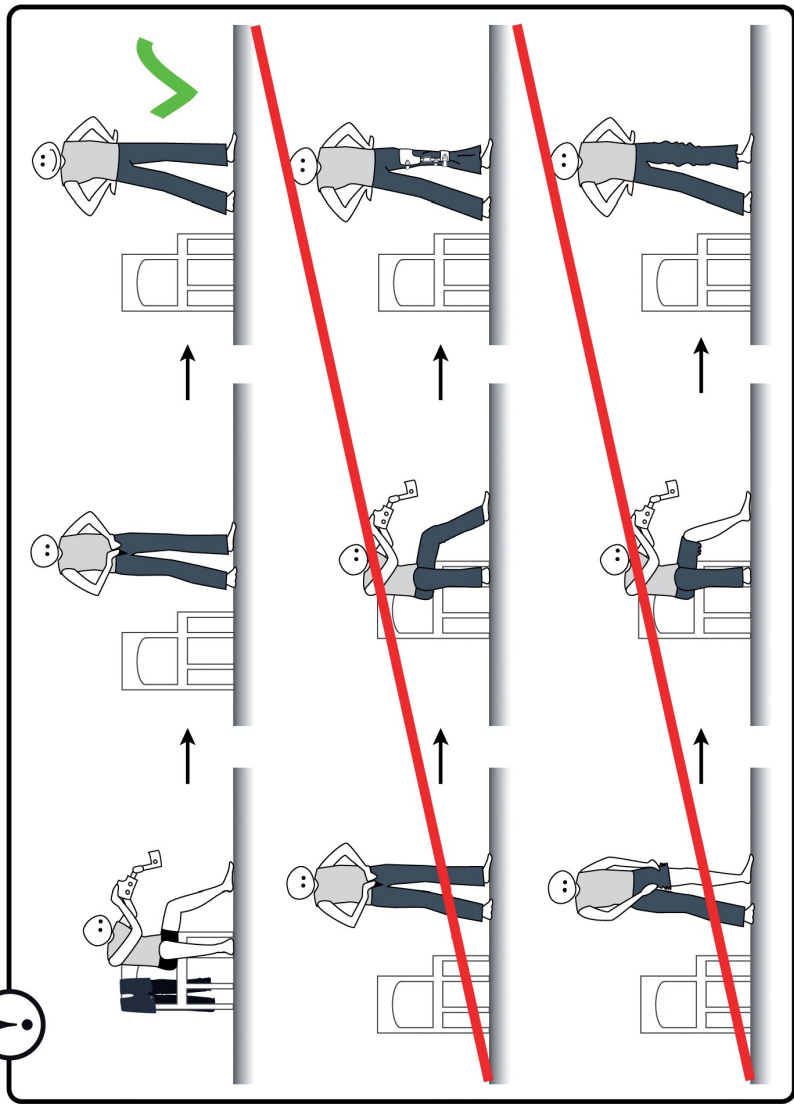

PROTEOR

6 rue de la Redoute - 21850 Saint-Apollinaire - France
Tél. 33 (0) 3 80 78 42 10 - Fax 33 (0) 3 80 78 42 15
ht.orthopedie@proteor.com - www.proteor.com

2H210

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ENTRETIEN

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Instructions for use

CARE OF THE ORTHOSIS

Wash the bands with a slightly damp sponge or cloth.
 The other mechanical components do not require particular care.
 The A.K. inside lining is removable and machine washable at 30°.
 The B.K. inside lining is removable and washable with warm water and soap.

STORAGE

Do not place the orthosis near a strong source of heat.
 Keep the orthosis out of the reach of pets.

USING CONDITIONS

The orthosis can be worn under any conditions, except for bathing.
 As far as possible avoid contact with sand or soil, because this could damage the functions of the joints.

As this orthosis is made to measure, it is convenient only for the patient it has been prescribed for. It must not be lent or re-used for other patients or other pathologies than the medial knee osteoarthritis.

POSSIBLE SIDE EFFECTS

Possible skin reactions, blotches, irritations, ...
 If the troubles remain, visit your doctor and/or CPO.

HYGIENE CONDITIONS

An everyday wash is necessary.
 The skin must be checked every day.
 The orthosis must be cleaned regularly.

CONTROLS BY YOUR DOCTOR AND/OR CPO

Keep in touch with the doctor who prescribed the orthosis and the CPO, so that the orthosis be checked all along the treatment.
 Keep to the recommended medical follow-up.
 In case of morphological changes, especially weight increase, do not hesitate to visit your CPO or your doctor.
 Before taking part in activities that can cause high stresses, contact the doctor who prescribed the orthosis to know if this is possible.

If the orthosis falls or sustains a shock, visit your CPO to check that the functions of the orthosis are not damaged. In any case do not try to repair or alter the orthosis yourself.

RECYCLING - END OF THE PRODUCT LIFE

The components of the orthosis are special waste. They must be returned to the manufacturer to be thrown away properly.

PROTEOR
 6 rue de la Redoute - 21850 Saint-Apollinaire - France
 Tél. 33 (0) 3 80 76 42 10 - Fax 33 (0) 3 80 76 42 15
ht.orthopedie@proteor.com - www.proteor.com

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